Multilink[®] Sprint

SPEED • SIMPLICITY • CONFIDENCE



Self-adhesive Conventional Resin Cement



SPEED

SIMPLICITY

CONVENTIONAL CEMENTATION WITH MULTILINK SPRINT IS FASTER...

Multilink Sprint is a self-adhesive, dual-curing resin cement indicated for all situations where conventional cementation is suitable. The system offers faster and easier processing than traditional conventional cements, with all of the advantages of resin-based cements.

- No etching with phosphoric acid
- No primers or bonding agents
- No hand mixing
- No mixing capsules
- No capsule activators, mixing devices or injectors
- No wait times (when light curing)

Multilink Sprint is packaged in a double-push syringe with an automix tip for direct application into the restoration without the need for additional accessories or mixing devices. Simply dispense the amount of cement desired with a consistent 1:1 mix every time.

CONVENTIONAL CEMENTATION WITH MULTILINK SPRINT IS EASIER...

Insertion of a zirconium oxide all-ceramic crown (IPS e.max ZirCAD) using Multilink Sprint.



Remove temporaries and clean prep.





Load restoration.

Seat and hold with firm pressure.






Remove excess cement.

Final result.

Clinical case by Dr. Thomas Seitner (Singen, Germany)

Multilink Sprint is indicated for the conventional cementation of indirect restorations made of:

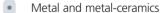



- High-strength all-ceramics (Zirconium oxide, Lithium disilicate, Aluminum oxide)
 - Fiber-reinforced composite







CONFIDENCE

CONVENTIONAL CEMENTATION WITH MULTILINK SPRINT IS STRONGER...

The acidic adhesion promoter (MDP) provides true self-adhesion on enamel, dentin and most restorative surfaces eliminating the need for any additional primers or bonding agents.


Cross-linking agent

Bonding agent

Compared to traditional conventional cements, Multilink Sprint offers many advantages associated with resin-based cements:


- Dual-cure (Self-curing and Light-curing initiators)
- Higher physical properties
- Adhesion to enamel and dentine
- Shade options and higher translucency
- Lower water solubility
- Radiopacity

Multilink Sprint is offered in three shades with varying degrees of translucency for enhanced esthetic options.

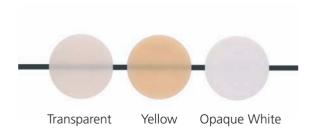


Based on the proven and successful formulations of Variolink® II and Multilink® Automix, Multilink® Sprint features material properties that are superior to traditional conventional cements and other competitive resin-based cements.


SHEAR BOND STRENGTHS (after light-curing)

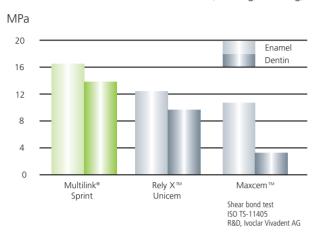

FLEXURAL STRENGTH (after light-curing)

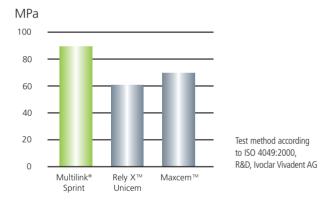

RADIOPACITY

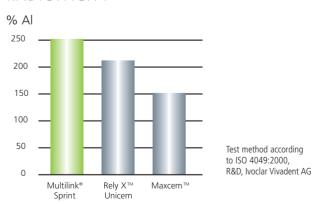

Multilink® is a registered trademark of Ivoclar Vivadent.
Rely X™ Unicem and Maxcem™ are not registered trademarks of Ivoclar Vivadent Inc.

Multilink[®] Sprint

DELIVERY FORMS





598180 Multilink Sprint Assortment

3 – syringes (9g each) Multilink Sprint

(1 each in shades: Transparent, Yellow, Opaque White)

45 - mixing tips

Multilink Sprint Refills

1 – syringe (9g) Multilink Sprint 15 – mixing tips



598178 Transparent 598181 Yellow

598182 Opaque White

592225 Multilink Mixing Tips Refill

15 – mixing tips

603018 Multilink Root Canal Tips Refill

5 – tips

Multilink Sprint is recommended for

≌e.max®

2 d.SIGN®

InLine®

⊭Postec®Plus

Cementation Selection Guide	Variolink® Veneer Adhesive, Total Etch, Light-cure	Variolink® II Adhesive, Total Etch, Dual-cure	Multilink® Automix Adhesive, Self Etch, Dual-cure	Multilink [®] Sprint Conventional, Self Etch, Dual-cure
Metal/Metal-based				
Inlays/Onlays	_	-	✓	√
Crowns/Bridges	_	_	1	✓
Endodontic Posts	-	_	✓	1
Maryland Bridges	_	ı	\	_
Glass Ceramics				
Inlays/Onlays	1	✓	1	_
Veneers	1	✓	-	-
Crowns	-	✓	1	-
3-unit bridges	-	✓	1	-
Oxide Ceramics (Zirconia, Alumina)				
Crowns/Bridges	_	_	1	1
Endodontic Posts	-	_	1	1
Indirect Composites (Fiber-reinforced)				
Inlays/Onlays	✓	1	1	_
Crowns	_	1	1	_
Endodontic Posts	_	1	1	1
✓ recommended				

United States

175 Pineview Drive Amherst, NY 14228 716.691.0010 800.533.6825 716.691.2285 Fax www.ivoclarvivadent.us

mail@ivoclarvivadent.us

Canada

2785 Skymark Avenue, Unit 1 Mississauga, Ontario L4W 4Y3 905.238.5700 800.263.8182 905.238.5711 Fax www.ivoclarvivadent.ca mail@ivoclarvivadent.ca



not recommended

