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NEO-TEC Dental

Carl-v.-Ossietzky-Str. 25, 99423 Weimar, Germany
www.neotecdental.de

SZ Reline Kit is designed as a kit to meet high requirements of modern fabrication of permanent soft relinings. The kit consists of **SZ Reline Primer**, **SZ Reline Cartridge (Automix)** and **SZ R-Varnish**.

SZ Reline Kit Cartridge is a long-term soft relining material based on addition curing vinyl silicones in auto-mixing cartridges. SZ Reline Cartridge is suitable for application on all synthetics based on polymethylmethacrylate and for a fast and easy fabrication of permanent soft relinings; chairside or indirect use in the dental laboratory. It meets the requirements of ISO 10139-2.

SZ Reline Cartridge relieves pressure areas in the upper and lower jaw. It provides the post-damming in case of adhesion problems and with flabby ridges: The position of the upper and lower dentures are fastened by increased adhesion. Sharp ridged alveolar processes are elastically covered.

SZ Reline Cartridge is also suitable for cushioning of implants and as overdentures during the healing process.

SZ Reline Primer is a **bonding agent** between SZ Reline Cartridge and the denture, and **SZ R-Varnish smoothes the rough trimmed surfaces** of the soft relining material.

1. Preparation of the Denture

Remove completely the old existing relining. Grind off the denture base approximately 1-2 mm. Prepare at the vestibular, lingual resp. dorsal borders a shoulder of approximately 2-3 mm in height with a depth of minimum 1 mm. Round off the borders and clean the denture thoroughly. Traces of grease have to be removed by means of pure medical alcohol and let dry.

2. Applying SZ Reline Primer for Adhesion

Apply SZ Reline Primer **uniformly and completely** onto the grinded and cleaned denture areas to which the material has to adhere. Let the primer dry for approximately **1 minute**. After use close the bottle immediately. Saliva, humidity or monomer liquids prevent the adhesion of SZ Reline Cartridge to the denture base.

3. Gun and Cartridge

Mount the cartridge into the application gun. Remove and dispose the closure-cap. For exact flow control extrude slightly material until uniform flow from both orifices is achieved. Install the mixing cannula onto the cartridge and extrude needed material.

Note:

Store used cartridge with fixed used mixing cannula. **Do not re-use the cartridge cap**. Closing the cartridge by turning on the cap between use could cause cross contamination of components and thus premature setting.

4. Relining of the Denture

4.1 Direct Chairside Relining Method

SZ Reline Cartridge has to be applied uniformly from the cartridge as described under point 3 onto the vestibular, lingual resp. dorsal borders. Then coat completely the remaining denture with a layer of minimum 2 mm. It is essential, that **all denture areas in contact with the gingiva** have to be coated.

After coating of the denture, **which has to be done for not more than 1:00 minute** (working-time at 23°C or 74°F) seat the coated denture into the patients mouth and let him close in normal occlusion. Let the patient carry out all functional and mimic movements. Setting is finished after approximately **4 minutes**. Then remove the relined denture and trim as described under point 5 and 6.

4.2 Indirect Method in the Laboratory

Invest denture to be relined into a flask with counter. Remove denture from the flask and prepare as described under point 1 and 2. Coat the model with an usual alginate based insulation. Extrude the required amount of SZ Reline Cartridge from the cartridge as described under point 3 and apply it uniformly onto the denture area to be relined. The layer of SZ Reline Cartridge should be approximately 1 mm more than the base material has been ground off.

Close flask with counter. Press with light pressure for approximately **30 minutes** (at 23°C or 74°F) or place flask in a pressure pot at **40-45 °C (104-113 °F)** for approximately **10 minutes**. Then deflask the relined denture and trim and polish as described under point 5 and 6.

Notes:

Do not use with standard (condensation reaction type) silicone impression materials.

EN Instructions for use

Do not use Latex gloves, they may interfere with the setting reaction of the impression material.

5. Trimming of the Denture

Smoothing of uneven spots: Remove excess material with a sharp scalpel or fine scissors. Then smooth rough areas and borders by means of a suitable grinding instrument (grinding sleeve for silicone materials) with approximately 8000 r.p.m. under low pressure.

6. Final Polishing of the Denture with SZ R-Varnish

For smoothing only of the trimmed functional borders use the enclosed **SZ R-Varnish**, which has to be used as a **1:1-mix of SZ R-Varnish Base** and **SZ R-Varnish Catalyst**: From the bottles of catalyst and base liquid, drop **5 drops** from both onto a well and mix it together with a spatula to a homogenous mixture and brush on the mixed liquid. Let dry at room temperature for approximately **5 minutes**.

7. Cleaning Hints for the Patient

Brush the relined denture several times a day under running water with a tooth brush and neutral soap. Cleaning bath: up to a maximum of 10 minutes.

8. SZ Reline Kit-Kombipackung

SZ Reline Primer	5 ml bottle
SZ Reline Cartridge	50 ml cartridge
SZ R-Varnish Base	5 ml bottle
SZ R-Varnish Katalysator	5 ml bottle
6 mixing cannulas	

9. Storage

Do not store above 25 °C (77 °F). Do not use after expiration date.

Warranty

The manufacturer warrants this product will be free from defects in material and manufacture. The manufacturer makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and manufactures's sole obligation shall be repair or replacement of the manufacturer.

Limitation of Liability

Except where prohibited by law, the manufacturer will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep away from children!
For dental use only!

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