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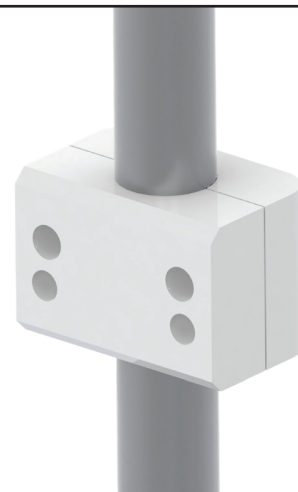
FIXE RUNDROHRKLEMME

UNIVERSELLE RUNDROHRKLEMME

ASSEMBLY INSTRUCTION

FIXED POLE CLAMP

UNIVERSAL POLE CLAMP

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PLEASE READ THIS INSTRUCTION CAREFULLY BEFORE INSTALLING!

Clarification of non-standardized Symbols:

 Warning of a danger point**AREA OF APPLICATION/ INTENDED USE**

CIM medical products are intended to mount and position medical electrical equipment (hereafter referred to as ME equipment) and accessories to those and are used in operating theatres as well as treatment rooms as auxiliary devices for doctors and medical staff.

The fixed pole clamps are exclusively to be used in conjunction with CIM support arms in the medical sector.

GENERAL NOTES

Before you want to mount the pole clamp to an existing pole, please verify that the pole can hold the weight. Please check the weight of the ME equipment to be mounted. The permissible total weight is mentioned on the corresponding data sheet.

WARNING Installation, Operation and Maintenance 

This section contains warnings regarding the installation, operation and maintenance of the pole clamp. This section must be read in its entirety before installing, operating and maintaining the pole clamp. Failure to follow these warnings may result in damage to equipment or injury to people.

Installation, operation and maintenance may only be carried out by qualified personnel. Assembly may only be done by qualified technical personnel. CIM med GmbH cannot be held responsible for any damage due to poor installation.

Product modifications, disassembling and reassembling and/or mounting any of the components in any other way than described below are permitted only after written approval by the CIM med GmbH construction and engineering department. CIM med GmbH cannot be held responsible for any damage or injury caused by product modifications and/ or amendments without prior permission by CIM med GmbH.

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GENERAL NOTES

⚠️ WARNING: EARTHING CONNECTION

The pole clamp is delivered with components' grounding. It is the healthcare institution's responsibility to connect the pole to the electrical grounding of the building. The arm is connected to the pole via the grounding screw of the adapter.

TOOLS REQUIRED

Torx wrench T45
Hex key 3.0
Hex key 5.0

CLEANING

The pole clamp may be cleaned with mild, non-abrasive solutions commonly used in hospital environment. Do not use steel wool or other abrasive materials. The product has been designed to the highest specifications to make cleaning as easy as possible. CIM med makes no claims concerning the efficacy of the disinfection processes.

It is recommended to test any cleaning solution on a small area not visible to verify compatibility. Please consult your hospital's infection control manager.

MAINTENANCE: MANNER AND FREQUENCY

Very little maintenance is required during the life span. Please check all screws on a regular basis (at least once a year) and tighten them if necessary.

In case of damage done to the load-bearing parts of the system it has to be strictly refrained from further usage of the mounting system.

SAFEKEEPING

For any questions that may arise at a later time please safekeep our manual!

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

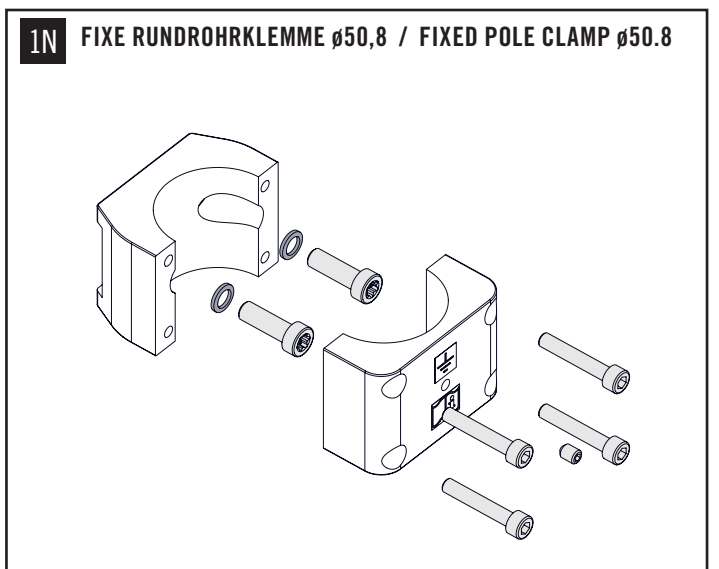
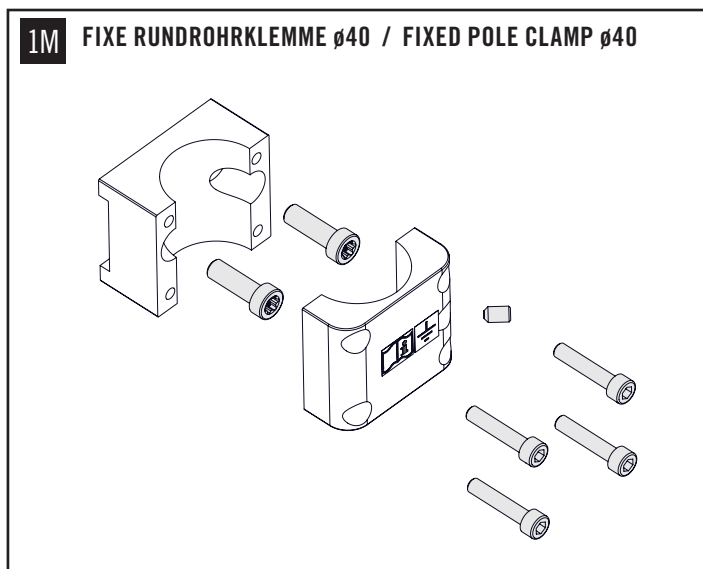
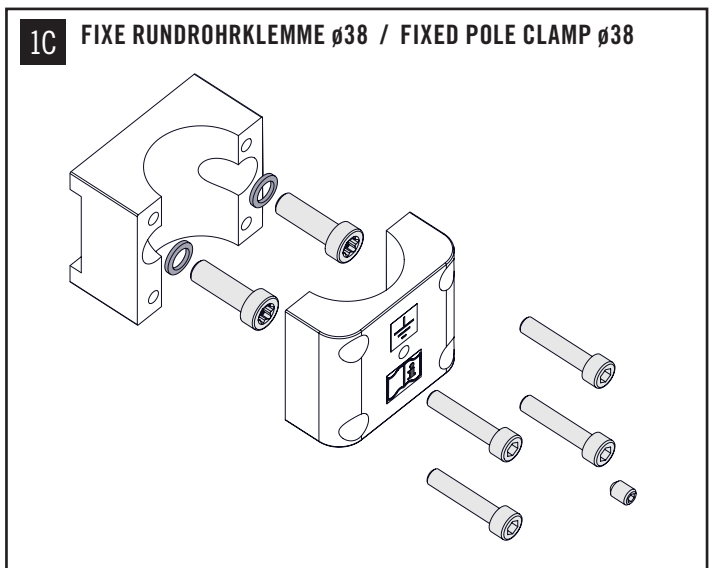
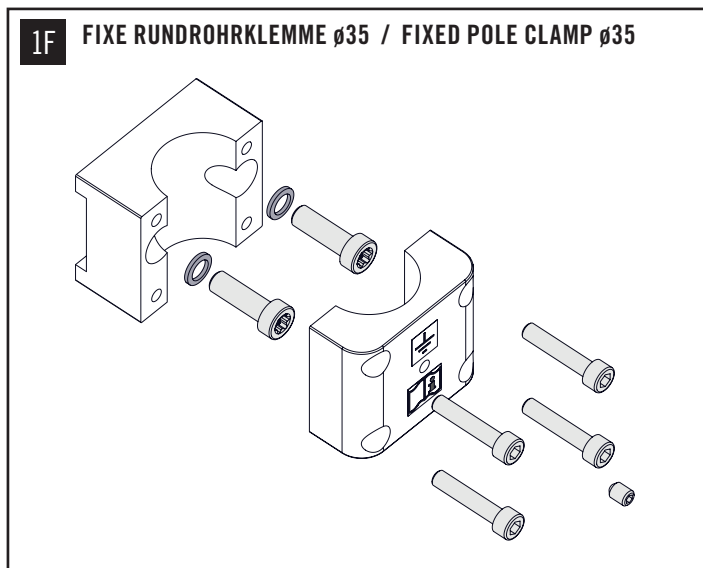
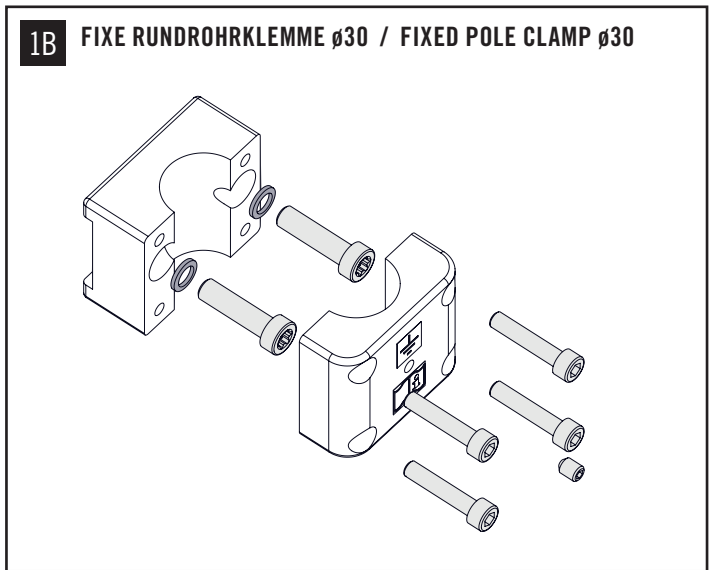
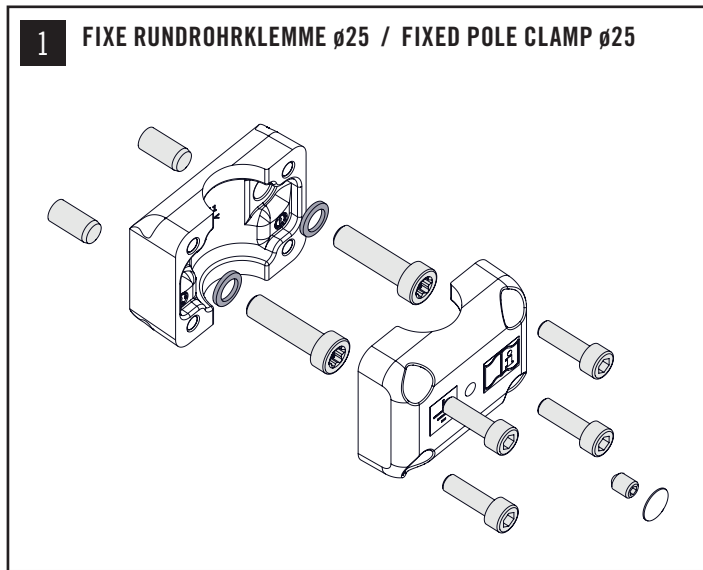
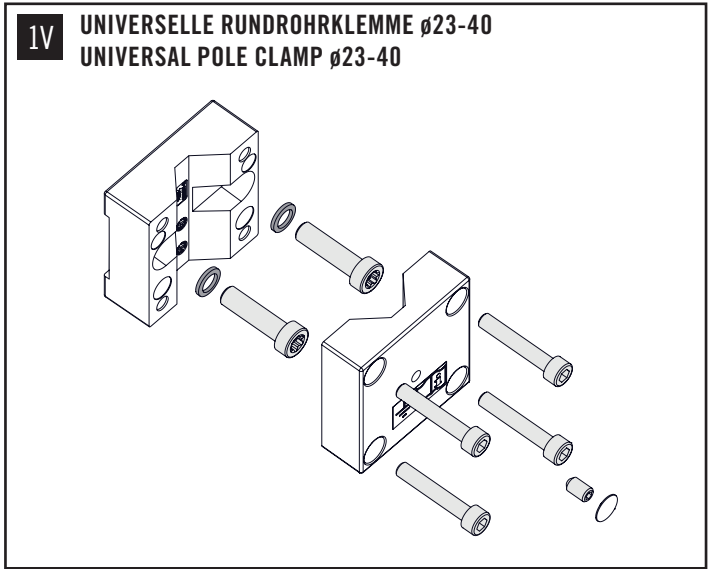
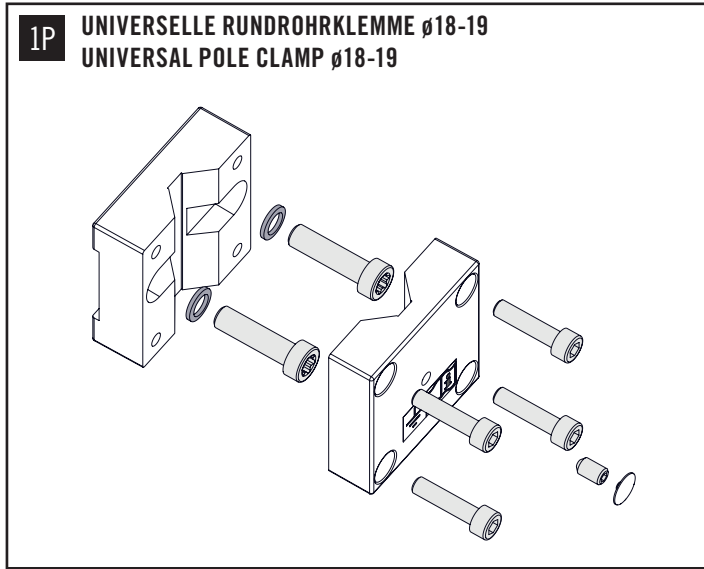
! CAUTION

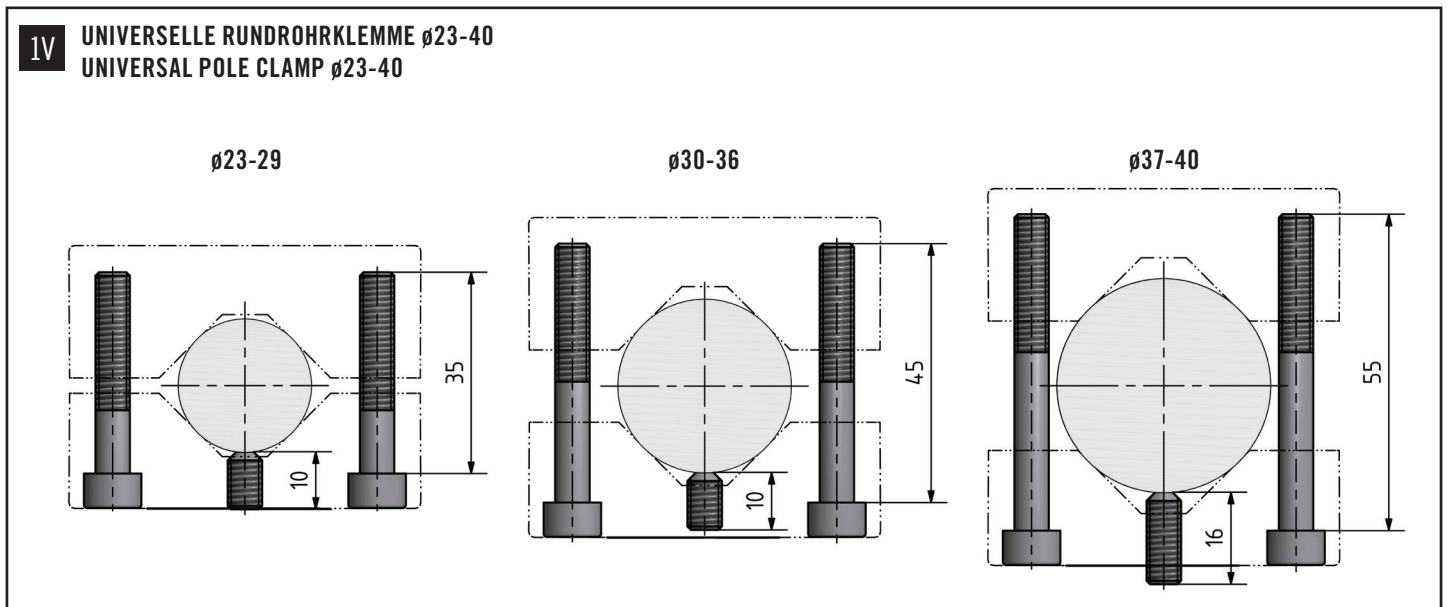The scope of delivery for the universal pole clamp $\phi 23-40$ consists of 3 different lengths of Allen key screws and 2 different lengths of earthing screws. Please refer to the diagram for information.

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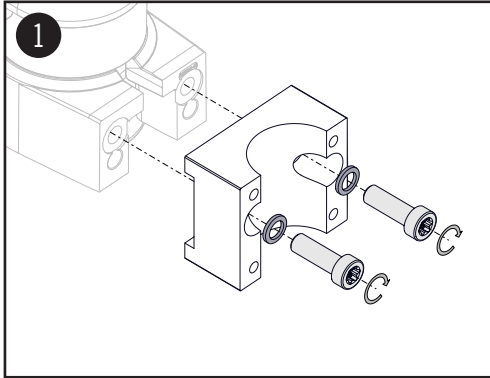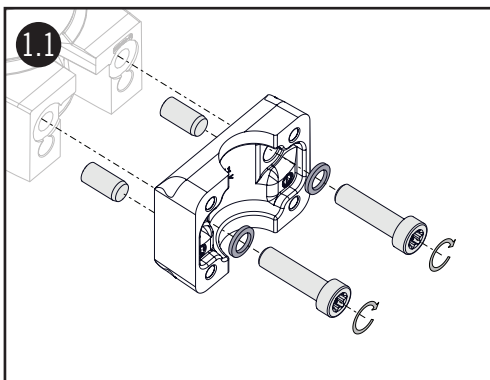
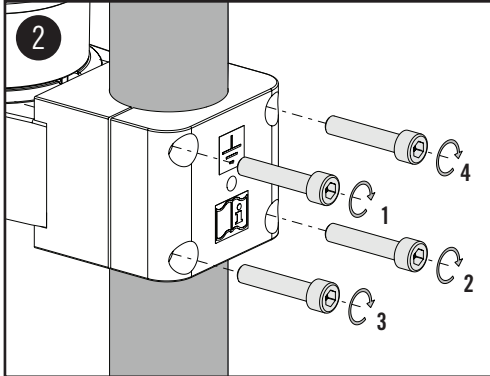
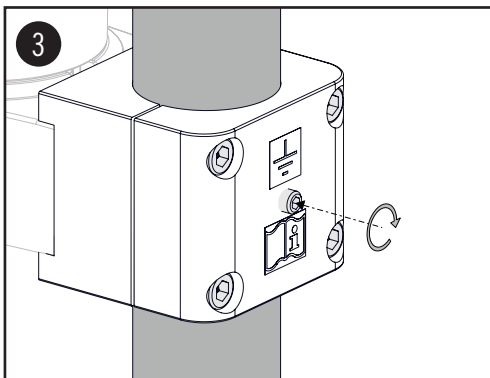
ASSEMBLY STEPS

Mount the adapter to arm using the 2 x M8 screws and detent washers.

Torque moment: 25 Nm (221 in-lbs)

! To guarantee a secure components grounding it is essential to use the both detent washers.

Fixed pole clamp $\varnothing 25$

For a secure fixation it is necessary to insert the two cylinder pins into the lower two holes.

Torque moment: 25 Nm (221 in-lbs)

Tighten the screws evenly to provide the best contact to the pole.

! CAUTION

The grounding screw must be tightened for a safe component grounding. If existant, cover the screw with the cover cap provided.