




Superior Implant Technology

AN UPDATE ON I10C:

CONICAL PLATFORM TRAPEZE IMPLANT

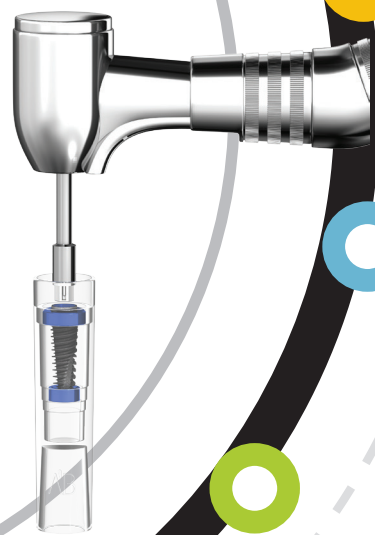

Each I10C implant is supplied and packaged with a single use, stainless steel screw carrier. This carrier is custom-made, in accordance with the implant geometry and design, offering a simpler, easier and safer procedure.

Instructions for use:

1

Open the implant package and connect the T5F tool to the contra angle

2

Grab the implant from the sterile vial with the T5F connected to the contra angle

3

Insert the implant to the bone until receiving primary stability not more than 30-35 RPM

4

Tighten the implant up to the bone level using T3 driver connected to ratchet T8 no more than 60NEWTON

5

Release the carrier from the implant with T1 driver

6

Use healing cap or cover screw for conical connection