
HiFocus™ Mid-Scala Electrode

Instructions for Locking the Stylet
into the
Insertion Tool


The following sequence should be followed to ensure the stylet is correctly locked into the HiFocus™ MS electrode insertion tool:


Remove the protective tube from the electrode insertion tool. Retract the slide of the electrode insertion tool and set the automatic stylet reset mechanism by pushing the button.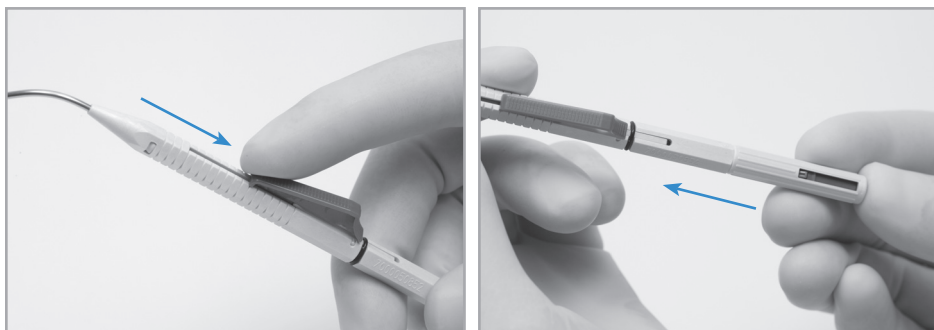


Hold the HiFocus™ Mid-Scala electrode by the protective sleeve. Avoid grasping the stylet. Do not remove the stylet from the HiFocus Mid-Scala electrode. Note the electrode 'jog' and the stylet head.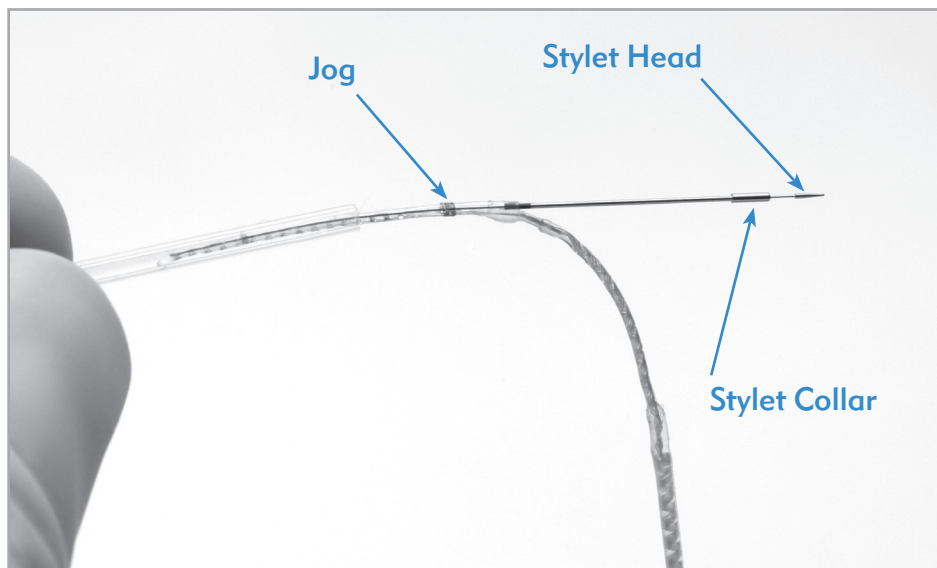


Align the electrode 'jog' to the engagement slot on the electrode orientation tube.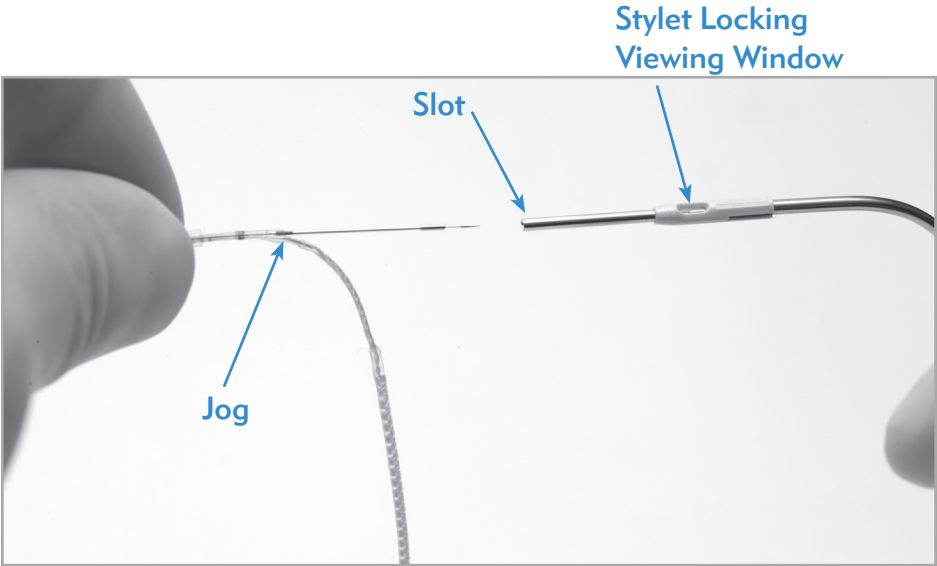

Insert the stylet fully until a slight 'click' is heard or resistance is felt. The stylet head should now be locked into the electrode insertion tool locking mechanism. This should be confirmed by viewing the locking mechanism under the microscope as shown below. The following images show the stylet properly engaged.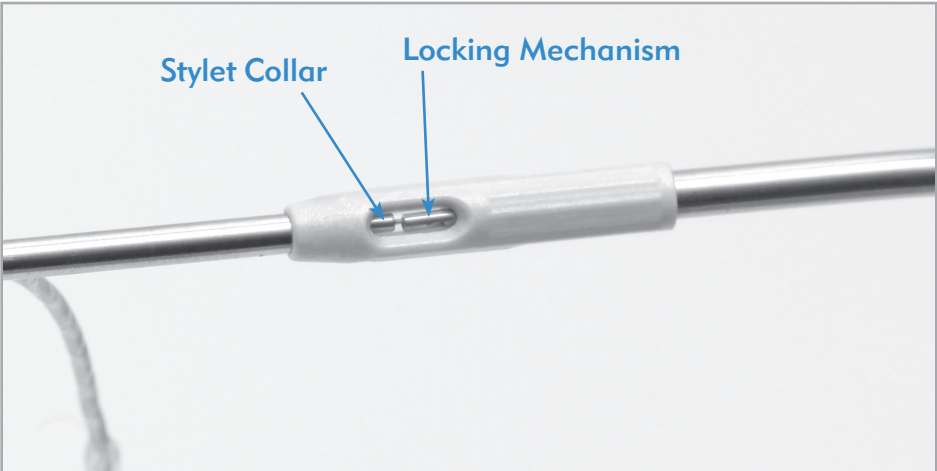

Locking Mechanism Tab

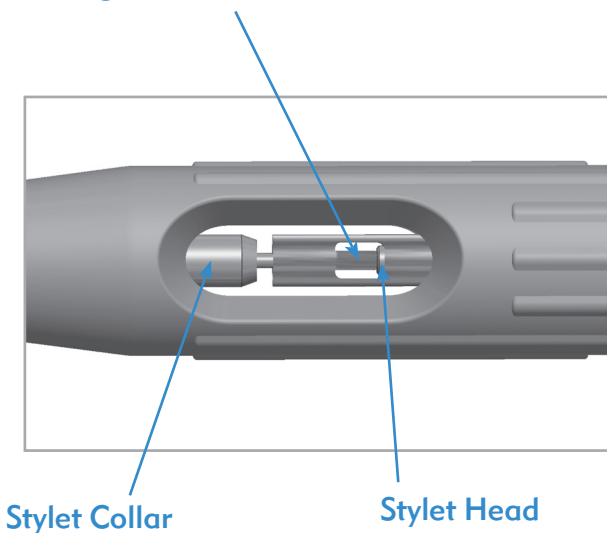The stylet head should be *fully beyond* the locking mechanism tab and the stylet collar should be visible in the stylet locking viewing window.

Remove the protective sleeve from the electrode taking care not to pull the electrode off the stylet. The electrode insertion tool is now ready to be used.

Note: *Once the stylet is locked into the electrode insertion tool it cannot be removed.*

If required, the electrode can be reloaded onto the stylet engaged in the electrode insertion tool, or onto the spare stylet provided in the implant package, by using the electrode reloading tool.

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© 2022 Advanced Bionics LLC. All Rights Reserved.
Advanced Bionics LLC
Advanced Bionics GmbH