



To request your  
**FREE**  
product sample,  
contact  
[Vernagel@vernacare.com](mailto:Vernagel@vernacare.com)



# Carrying bodily fluids is no longer high risk.

A sachet of **VernaGel** solidifies up to 1.2 litres of bodily fluids in seconds, reducing the risk of spills and infection.

## What is it?

VernaGel is a dry granule with rapid absorption properties. VernaGel stabilises bodily fluids and other medical waste in seconds, transforming them into an easily transportable, semi-solid gel.

### Why VernaGel?

-  Prevents spillages & accidents
-  Helps reduce the spread of infection
-  Minimizes & eliminates residual odours
-  Improves patient and healthcare worker dignity
-  Reduces nurses workload
-  Saves laundry costs



**VernaGel is a revolutionary product exclusively supplied by Vernacare and used worldwide.**

## How does VernaGel work?



**1.** Place one sealed VernaGel sachet into a container.



**2.** Add bodily fluid.



**3.** Within seconds, VernaGel will react and turn up to 1.2 litres of bodily fluids into a semi-solid gel.

## Where can I use VernaGel?

VernaGel requires no special skills or training to use – making it ideal to use in almost every kind of healthcare situation where fluids are present.

- Urinals, bedpans, slipper pans, vomit bowls and washbowls of any material
- Sharps residues
- Spillages
- Liquid medicine disposal
- Colostomy bags
- Sanitary waste

## Where can VernaGel be disposed?

- Disposal units
- Clinical waste
- Sharps bins
- Toilet
- Slop hopper



**DO NOT** tip 'gelled' substance into wash basins or sinks.



**DO NOT** eat.  
**DO NOT** leave patient unattended.

**Formats available**



**VernaGel sachets**  
100 sachets

Product Code: 450SC100



**VernaGel Bottle**  
Loose powder

Product Code: 456MA475



**VernaGel Large Tub**  
Loose powder with scoop

Product Code: 456MA400

**Vernacare**

1 Western Avenue,  
Matrix Park,  
Chorley PR7 7NB, UK  
**T:** +44 (0)1204 529494  
**E:** [vernagel@vernacare.com](mailto:vernagel@vernacare.com)  
**www.vernagel.com**