

Stallergenes Greer offers a portfolio of allergy testing and treatment for your patients

MAKE THE MOST OF ALLERGY TESTING WITH **SKINTESTOR OMNI™**



WHETHER YOU'RE ALREADY SKIN TESTING OR LOOKING TO START

Skintestor OMNI™ gives you confidence with accurate results.¹



97.2%

POSITIVE Predictive Value¹

Percentage of **true positive**
compared to total positive results



86.2%

NEGATIVE Predictive Value¹

Percentage of **true negative**
compared to total negative results



Prick skin test studies were conducted at three clinics in 2010. The Skintestor OMNI was loaded with a positive control (histamine) and a negative control (glycerol-saline solution)¹:


- True positive reactions were defined as diameters 2 mm or higher relative to those of negative controls per subject
- True negative reactions were defined as 0 mm wheal diameters or 0-5 mm erythema diameters
- Results included both wheal and erythema reactions produced on the volar surface of subjects' arms by glycerosaline negative control and histamine positive control solutions loaded onto each device

Please see the pocket for the Skintestor OMNI™ Usage Guide.

MAXIMIZE YOUR ALLERGENIC EXTRACTS WITH SKINTESTOR OMNI™



Conduct 1,031 tests, on average, per 5 mL vial of glycerinated extract^{2*†}

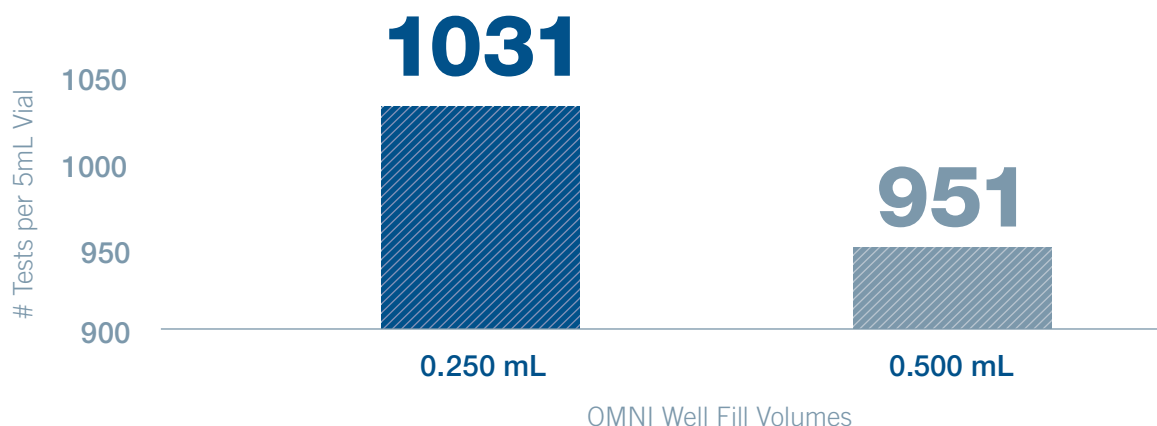

*When filled to optimal level= 0.250mL²

†Results of glycerinated extract stored at room temperature (20-25°C)

In 2010, Stallergenes Greer conducted gravimetric volume retention tests for the Skintestor OMNI²:

- Glycerol-saline diluent was tested at both room temperature (20-25°C) and refrigerated (3.8-5.2°C)
- All measurements were performed 25 times at room temperature
- Yields were calculated from the average weights and corresponding volumes removed from the wells at each fill level.



For more information about Stallergenes Greer skin testing devices or to schedule a device demonstration, contact a Stallergenes Greer Customer Care Specialist at

800.378.3906 or email customercare@greerlabs.com.

SKINTESTOR OMNI™

ENABLES QUICK AND EFFICIENT ALLERGY TESTING


The Skintestor OMNI™ System includes the self-loading, multiple-site skin testing device and additional testing equipment.

The Skintestor OMNI System's unique features help meet the needs of your practice and patients.



10 test sites per applicator

More tests in less time

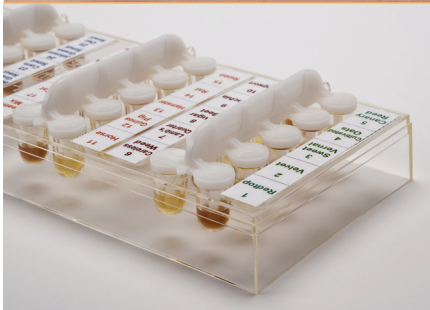

Large covered wells

Designed to keep allergens from mixing to maintain the integrity of the extract



Self-loading

Designed to be easy to fill

The versatile design of the Skintestor OMNI allows for simple use, whether skin testing on a patient's arm or back.



GET THE MOST OUT OF SKINTESTOR OMNI™ WITH OUR SELECTION OF EXTRACTS AND SUPPLIES



Compact design of well tray

Takes up less refrigerator space

Choose between 20 or 40 well labeled trays

Gives you flexibility to customize a system that works best for your practice



Wide range of allergenic testing extracts

To meet the needs of your patients

Please see the pocket for the Skintestor OMNI™ Usage Guide.



ACQUIRE THE SKILLS TO
**USE OUR SKIN TESTING
DEVICES WITH CONFIDENCE**

Stallergenes Greer allergy immunotherapy (AIT) professionals lead hands-on, personalized sessions to teach you and your staff the proper testing protocols to achieve reliable skin testing results.

The GREER® Skin Testing Proficiency Program offers:



Live Presentations

led by one or more of our allergy immunotherapy professionals



Pre- and Post-Evaluation Forms

to help assess participants' skin testing abilities before and after training



Skin Testing Proficiency Instruction Brochure

with easy-to-follow, step-by-step directions and protocols for achieving proficiency with our skin testing device, Skintestor OMNI™



Recognizing Proficiency

Acknowledgements for demonstrating proficiency and the ability to train others

For more information, call your Customer Care Specialist at 800-378-3906.

Please see the pocket for the Skintestor OMNI™ Usage Guide.

SKINTESTOR OMNI™

MULTIPLE ALLERGENS, ONE DEVICE



Quick and efficient multi-site testing:

- 10 test sites per applicator
- Large covered wells
- Choose between 20 or 40 well trays
- Self-loading

For more information about Stallergenes Greer skin testing devices or to schedule a device demonstration, contact a Stallergenes Greer Customer Care Specialist at

800.378.3906 or email customercare@greerlabs.com.

Please see the pocket for the Skintestor OMNI™ Usage Guide.

References: 1. Data on file. GREER. DS082710: *Summary of OMNI Skin Test Studies with Histamine Positive and Diluent Negative Controls.* August 2010. 2. Data on File. GREER. DS043010: *Comparisons of uptake of 50% glycerosaline by Skintestor OMNI devices using wells loaded with 0.250 mL and 0.500 mL fills at ambient and refrigerator temperatures.* April 2010.

Stallergenes Greer PO Box 800, 639 Nuway Circle NE,
Lenoir, NC 28645 800.378.3906/www.greerlabs.com
© Stallergenes Greer 2017. All Rights Reserved. 121916H2105 12.16

STALLERGENES  **GREER**

USAGE GUIDE FOR SKINTESTOR OMNI™

STEPWISE INSTRUCTIONS FOR TESTING WITH THE DEVICE*:



- 1. MATCH ARROW** on the Skintestor OMNI™ device to the arrow on the tray
- 2. INSERT DEVICE** into the tray and remove
- 3. HOLD THE DEVICE PERPENDICULAR** to the skin with arrow pointing toward the patient, allowing for easy location of each test site
- 4. PRESS DOWN** with moderate pressure
- 5. WITHOUT LIFTING, ROCK THE DEVICE (MOVE ACROSS THE TEST SITE WITH EQUAL DISTRIBUTION OF PRESSURE)**
 - On arm: rock the device from side to side
 - On back: rock the device from side to side, then top to bottom
- 6. WITHDRAW** the Skintestor OMNI (When properly performed, the test will leave visible marks on the skin without causing bleeding)
- 7. DISCARD** the used device into a suitable biohazard container
- 8. REPEAT** steps until testing is complete

*Skintestor OMNI Directions for Use.